





BIOCREDIT

COVID-19 AG antigen rapid Test



BIOCREDIT COVID-19 Ag rapid test kit

Intended Use Detection of SARS-CoV2 antigen

Package 20 Tests / kit

Storage 1~40℃

Specimen Type Nasopharynx / Nasopharyngeal
Shelf life 24 months from manufacture date

Time to result 5~8 minute

Click to watch Video -> RAPIGEN BIOCREDIT AG RAPID TEST
*HOW TO USE *WHAT IS INCLUDED *INDIVIDUAL PACKED



Registration No. FDA PH-0469840



DOH R.I.T.M VALIDATED

Specimen type

- BIOCREDIT COVID-19 Ag Test: Nasopharyngeal SWAB





Performance Characteristics

| Europe | | PCR (after symptoms occur) | | Sensitivity | Specificity |
|-------------|----------|----------------------------|----------|-------------|-------------|
| | | Positive | Negative | | |
| BIOCREDIT | Positive | 24 | 0 | | |
| COVID-19 Ag | Negative | 1 | 25 | 96.0% | 100.0% |
| Total | | 25 | 25 | | |

| South America | | PCR (after symptoms occur) | | Sensitivity | Specificity |
|---------------|----------|----------------------------|----------|-------------|-------------|
| | | Positive | Negative | | |
| BIOCREDIT | Positive | 10 | 0 | | |
| COVID-19 Ag | Negative | 1 | 109 | 90.9% | 100.0% |
| Total | | 11 | 109 | | |

| Total | | PCR (after symptoms occur) | | Sensitivity | Specificity |
|-------------|----------|-------------------------------|----------|-------------|-------------|
| | | Positive | Negative | | |
| BIOCREDIT | Positive | 34 | 0 | | |
| COVID-19 Ag | Negative | 2 | 134 | 94.4% | 100.0% |
| Total | | 36 | 134 | | |





Cover acute to convalescent phase of SARS-CoV-2 infection with BIOCREDIT COVID-19 AG/AB Test


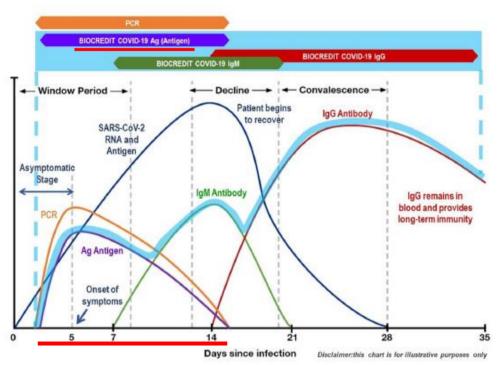


Figure 1: Variation of the Levels of SARS-CoV-2 RNA and Antigen, IgM and IgG after infection.

BLACK GOLD PARTICLE - BEST QUALITY IN THE WORLD



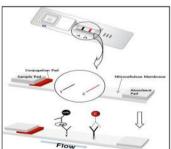

Strong Point

- · High Sensitivity (More than 4-32times Best Quality in the world)
- Easy to inspect using dual color lines (Control: Red, Test: Black)
- · High repeatability and stability using gold conjugation
- · High concentration manufacture system





Recommended BIOCREDIT COVID-19 RAPID Test system

Fever or Respiratory Symptoms
[RAPID RESULT NEEDED]

COVID 19 AG TEST

[Priority Recommended]

COVID 19 AB TEST

[Recommended with AG]

- AG TEST RESULT POSITIVE NEED PCR TEST [MOST INFECTIOUS Period]
- AB TEST RESULT POSITIVE NEED PCR TEST
- BOTH TEST RESULT POSITIVE NEED PCR TEST [RED ALERT]

PCR TEST

BOTH AG/ AB - RESULT CONFIRMED NEGATIVE [SAFE from ALL TITER]

NEGATIVE CASE

Official Representative Philippines:

Medical@wwlgroup.biz / www.wwlifelink.com

(632) 5310-9184 loc.321/316 . 0917 135 4416 / 0977 820 8800